



Multi-arm Sling Instruction for Use

Model ref.: PAS
Product code: PAS/U



Intended Use

The Multi-Arm Sling is designed to provide immobilisation of the arm/elbow/shoulder

Indications for Use

The Multi-Arm Sling is recommended for use by individuals experiencing any of the following conditions:

- Shoulder/Arm Trauma
- Post-operative shoulder care
- Shoulder surgery
- Fracture management of stable clavicle and shoulder fractures
- Shoulder subluxation/Dislocation

Contraindications

The Multi-Arm Sling must not be used:

- for treatment of individuals with severe fixed deformity
- over open wounds that may come in to contact with the Multi-arm sling

Symbols Glossary



Consult instructions for use



Caution



Batch code



Manufacturer



CE logo

General Instructions

-   The Instructions for Use must be read and understood before attempting to use the device.
-  It is recommended that the Multi-Arm Sling be fitted by an appropriately trained medical professional.

- 

 Care should be taken to instruct the user and carer on the correct fitting procedure as shown under “Fitting Instructions”.
- 
 Do not use if any damage is apparent

Available sizes

Product Code	Description	Basic UDI-DI	Device Identifier UDI
PAS/U	Multi-arm Sling	50552043PASS7	5055204306262

Components Included

Description	Quantity
Multi-arm sling	1

Fitting Instructions

Step 1 – Position the sling and shoulder strap on the arm and pull tight around the elbow. Adjust for length by turning the sling inwards from the patients fingers to the desired position.



Step 2 – Close the sling over the hand by attaching the D-ring strap.



The centrally placed ring must always be located on top of the arm.

The position of the second ring, either to the front or back of the sling is optional, allowing a choice of attachment position for the immobilisation strap.

Step 3 – Close the sling around the elbow by attaching the sling closure strap.



Step 4 – The arm can now be supported in the desired position by bringing the shoulder strap round the back, over the shoulder and fixing through the top D-ring.



Step 5 – To immobilise the shoulder, simply attach one end of the immobilisation strap at the elbow, take around the patients back...



Step 6 – ...and fix through the D-ring provided. The strap can be infinitely adjusted to provide a secure and comfortable fitting.



When to Consult a Healthcare Professional



The Multi-arm Sling should be removed if any of the following issues are experienced:

- Discomfort
- Skin irritation

Care and Cleaning Instructions

- No specific servicing or maintenance is required.
- The user should visually inspect the integrity of the device before each use. If any damage is apparent the device must be replaced.

- Periodically the device which may worn next to the skin may require washing (see details below).
- Hand wash only at low temperature (40 degrees Celsius) using a mild detergent and leave to air dry.
- Do not use bleach.
- Do not tumble dry.
- Do not iron.
- Do not machine wash

Spare and Accessories

For spares and accessories please visit shop.talarmade.com

Additional Guidance and Training

For additional guidance and training resources please contact www.talarmade.com

Materials

Nylon 60%, Foam 30%, Cotton 5%, PVC 5%

Storage

When not in use store in original packaging away from UV light

Disposal

Dispose as general waste. Disposal must comply with applicable local and legal regulations.

Terms and Conditions

For terms and conditions please visit www.talarmade.com

Contact Information

Address: Talar Made Ltd. Edinburgh House, Millennium Way, Chesterfield, UK., S41 8ND

Tel: 01246 268456

Fax: 01246 268648

Web: www.TalarMade.com

Email: for orders: orders@talarmade.com, or, for more information: info@talarmade.com