

Prolevo Wedge

Fitting & Use Instructions



Scan to view a fitting video



Ordering Information

Style	Pump	Qty	Size(cm)	Order Code	NHS SC Code
Inflatable	Yes	1	83x61	PWP-UC-IP-OS	TLC85202

The Prolevo Wedge is for multi-patient use

Step 1



Inspect the device to ensure it is clean and undamaged. Ensure the attachment method to the bed frame or mattress platform is secure. Explain the procedure to the patient where appropriate.

Step 2



Inflate the product by inserting the pump through the blue inflation valve. Inflate the device until the material is taut. Do not over inflate. Seal the blue inflation valve and push valve into product.

Step 3



Remove the cap from the exhaust valve (attached via the red mount). Leave valve open for 30 seconds (it will automatically exhaust excessive air pressure). Reconnect the valve cap.

Step 4



Position the Wedge on the bed, ensuring the largest air cell is facing towards the end of the bed and the exhaust valves are pointing down. Position the Prolevo Wedge to ensure the heels extend beyond the end of the device.

Step 5



Fasten the straps under the bed frame or mattress to secure the device. Apply a bed sheet over the device prior to use ensuring the sheet conforms to the full shape of the Wedge and bed. Do not over tension bed sheet beyond the Prolevo Wedge.

Step 6



Gently place the patient's lower legs onto the wedge so the calves are supported. Ensure the heels extend beyond the end of the wedge and remain suspended in the air. Confirm that no part of the heel is contacting the mattress or bed surface.



+44 (0) 1246 268456



prolevo@talarmade.com



prolevo.com

By **TalarMade**